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TECHNICAL DATA SHEET

PRODUCT: **BLOOD AGAR – CHOCOLATE AGAR**
REFERENCE: **020136**

Date 1st Edition:
7th 2009
Date 4th Revision:
6th 2024

DESCRIPTION

A bifurcated plate with BLOOD AGAR – CHOCOLATE AGAR for the cultivation of all aerobic bacteria and identification of hemolytic Gram (+) cocci (BLOOD AGAR). – For the cultivation of microorganisms with special nutritional requirements, such as *Haemophilus* and *Neisseria* species (CHOCOLATE AGAR).

PRINCIPLE OF THE METHOD

The Blood Agar Base contains high-value nutrients that provide vitamins, carbohydrates, and other organic compounds. Sodium chloride supplies the necessary minerals and maintains osmotic balance and electrolyte equilibrium. The addition of 6% horse blood enhances the nutritional quality of the medium. Additionally, the hemolysis and the type of hemolysis of horse red blood cells assist in the identification of certain Gram (+) cocci.

FORMULA BLOOD AGAR	g/litre
Columbia Peptone Mixture	25.1
Corn Starch	1.0
Sodium chloride	5.0
Agar No. 2	12.0
Horse Blood	60ml

Appearance: Red – burgundy, opaque, due to the addition of blood.

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Chocolate Agar is an enriched medium used for qualitative procedures, for the isolation and cultivation of pathogenic microorganisms, particularly *Neisseria* species and *Haemophilus* species from various clinical samples.

FORMULA CHOCOLATE AGAR	g/litre
Columbia Peptone Mixture	23.0
Corn Starch	1.0
Sodium chloride	5.0
Agar No. 2	12.0
Horse Blood	60ml

Appearance: Brown – chocolate-colored, non-transparent due to the addition of blood.

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STORAGE AND TRANSPORT CONDITIONS

The petri dishes must be stored at **2 – 12 °C** inside their packaging until the time of use. Prolonged storage at temperatures below **2 °C** creates excessive moisture inside the material, increasing the risk of contamination. Even momentary freezing destroys the material. Excessive heating should also be avoided. The petri dishes can be used until the expiration date indicated on the label. Stability studies have shown that, for transport, petri dishes can remain at **6 - 25 °C** for **4 days** or at **25 - 40 °C** for **48 hours** without affecting the product's performance.

USAGE INSTRUCTIONS

Inoculate the petri dishes using the isolated colony streaking technique.

Incubate at $35\text{--}37^{\circ}\text{C}$ under microaerophilic conditions with 5 – 10% CO_2 and examine the cultures after overnight incubation (initial phase) and again at approximately 48 hours (final phase).

INTERPRETATION OF RESULTS

BLOOD AGAR:

Streptococcus pyogenes forms small, gray-white colonies with β -hemolysis.

Streptococcus pneumoniae forms small, flat colonies with α -hemolysis.

Staphylococcus aureus forms relatively large colonies (1-3 mm at 24 hours, 3-8 mm after 5 days of incubation).

Their color ranges from cream-yellow to orange, depending on incubation time.

It may cause α or β hemolysis.

CHOCOLATE AGAR:

Haemophilus influenzae forms small, moist colonies with a slight odor.

Streptococcus pneumoniae forms small, flat colonies with a greenish color.

Neisseria meningitidis forms large, gray-blue colonies.

Neisseria gonorrhoeae forms small, gray-white, colorless, mucous-like colonies.


METHOD LIMITATIONS

Final identification must be performed using biochemical and serological tests (e.g., Latex Agglutination Test), which can be carried out directly from suspected colonies.

QUALITY CONTROL

BLOOD AGAR

Microorganism	ATCC	Growth	Haemolysis
<i>Escherichia coli</i>	25922	Good	-
<i>Pseudomonas aeruginosa</i>	27853	Good	-
<i>Staphylococcus aureus</i>	25923	Good	B
<i>Streptococcus pneumoniae</i>	6303	Good	A
<i>Streptococcus pyogenes</i>	19615	Excellent	B

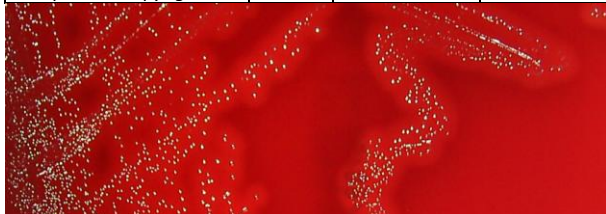Streptococcus pyogenes ATCC 19615

CHOCOLATE AGAR

Microorganism	ATCC	Colonies
<i>Haemophilus influenzae</i>	10211	Small, moist, with a slight odor
<i>Streptococcus pneumoniae</i>	6305	Small, flat, displaying a green color.
<i>Neisseria meningitidis</i>	13090	Large colonies, gray blue in color.
<i>Neisseria gonorrhoeae</i>	43069	Small, gray, white, colorless, mucous-like.

Haemophilus influenzae ATCC 10211

WASTE DISPOSAL OF WASTE

Materials that show no growth can be considered as non-hazardous waste and disposed of accordingly. Materials that show colony growth must be disposed of according to the guidelines for infectious or potentially infectious waste. The laboratory is responsible for the proper management of infectious waste according to its nature and level of risk and must handle and dispose of it (or assign its management and disposal) in compliance with the applicable regulations.

SPECIFICATIONS

BLOOD AGAR – CHOCOLATE AGAR - CE

ITEM	CODE	PACKAGE	STORAGE	SHELF LIFE
Divided Petri Dish 9cm	020136	10 pieces	2 – 12 °C	2 months

Produced in Greece by the company Bioprep in accordance with the requirements of the European Directive 2017/746.

BASIC UDI-DI: 5212037714010401WF. EDMA (14 01 04 01) non-chromogenic media (Plates).

The Bioprep company has been certified according to the standards: EN ISO 9001:2015 / EAOT EN ISO 13485:2016 DY8d/1348/2004

LITERATURE REFERENCES

BLOOD AGAR

Ellner, P.D., Stoessel, C.J., Drakeford, E. and Vasi, F. (1966). A new culture medium for medical bacteriology. Amer J. Clin Pathol. 45. 502-504.

American Public Health Association (1950). Diagnostic Procedures and Reagents. 3rd edn. A.P.H.A., New York.

CHOCOLATE AGAR

Sng, E.H., Rajan, V.S., and Lim, A.L. (1977) Simplified media for isolating *Neisseria gonorrhoeae*. J. Clin. Microbiol., 5 (4), 387.

Sottnek, F. O., Biddle, J. W., Kraus, S. J., Weaver, R. E., And Stewart, J. A. (1980) Isolation and Identification of *Haemophilus ducreyi* in a clinical study. J. Clin. Microbiol., 12 (2), 170.

Lewis, J. S., and Wiesner, P.J. (1980) Gonorrhea: Current laboratory methods. Lab. Management, Sept., p.33.

IN VITRO MANUFACTURER'S DATA

G. PAPANIKOLAOU & CO

PRODUCTION LABORATORIES OF CULTURE MEDIA

Potamou 5, Industrial Area Keratea, Attica

P.O. Box: 4893, Postal Code: 9001 - Tel: +30 2299066113. Fax: +30 2299066112
E-mail: bioprep1@otenet.gr www.bioprep.gr