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BLOOD AGAR

Ellner, P.D., Stoessel, C.J., Drakeford, E. and Vasi, F. (1966). A new culture medium for medical bacteriology. Amer J. Clin Pathol. 45. 502-504.
American Public Health Association (1950). Diagnostic Procedures and Reagents. 3rd edn. A.P.H.A., New York.

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E-mail: bioprep1@otenet.gr www.bioprepare.gr

TECHNICAL DATA SHEET

PRODUCT: **BLOOD AGAR – CHOCOLATE BACITRACINE**
REFERENCE: **020303**

Date 1st Edition:
7th 2009
Date 4th Revision:
6th 2024

DESCRIPTION

Dichotomous BLOOD AGAR – CHOCOLATE BACITRACIN petri dish for the cultivation of all aerobic bacteria and the identification of hemolytic gram-positive cocci (BLOOD AGAR). – For the cultivation and isolation of *Haemophilus influenzae* (CHOCOLATE BACITRACIN).

PRINCIPLE OF THE METHOD

Blood Agar Base contains highly nutritious components that provide vitamins, carbohydrates, and other organic elements. Sodium chloride supplies essential minerals and maintains osmotic and electrolyte balance. The addition of 6% horse blood enhances the nutritive properties of the medium. Furthermore, hemolysis and the type of hemolysis of horse red blood cells aid in the identification of certain gram-positive cocci.

FORMULA BLOOD AGAR	g/litre
Columbia Peptone Mixture	25.1
Corn Starch	1.0
Sodium chloride	5.0
Agar No. 2	12.0
Horse Blood	60ml

Appearance: Red to burgundy, opaque due to the addition of blood.

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CHOCOLATE BACITRACIN is an enriched medium designed for the isolation and cultivation of *Haemophilus* species from a variety of clinical samples. The interaction with bacitracin facilitates the selective isolation of *Haemophilus* species. Bacitracin is a polypeptide antibiotic that inhibits gram-positive bacteria and *Neisseria* species.

ΣΥΝΘΕΣΗ CHOCOLATE BACITRACINE	g/litre
Columbia Peptone Mixture	23.0
Corn Starch	1.0
Sodium chloride	5.0
Agar No. 2	12.0
Bacitracine	75mg
Horse Blood	70ml

Appearance: Brown to chocolate, opaque due to the addition of blood.

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PRECAUTIONS

To BLOOD AGAR – CHOCOLATE BACITRACINE is an in-vitro laboratory diagnostic material and should only be handled by qualified people in the laboratory. This material contains peptones and extracts of animal origin. The certificates regarding the origin and health status of the animals do not fully guarantee the absence of transmissible pathogens. For this reason, it is recommended that these materials be treated as potentially infectious, with the usual safety precautions (avoiding ingestion or inhalation). Plates should always be handled with gloves and in Laminar flow Class II, to avoid contamination mainly by saprophytic fungi. If the plate is cracked or the bag has a hole, do not use it. Do not use petri dishes if there are signs of microbial contamination. The thickness of the agar must be 4 - 5 mm and the material without cracks, dryness or other signs of deterioration. After the expiry date the material is unfit for use. In case of contact with the skin, wash immediately with plenty of water and soap. Positive samples must be destroyed according to the hygienic rules prescribed for the management of contaminated samples.

STORAGE AND TRANSPORT CONDITIONS

The petri dishes should be stored at **2–12 °C** inside their packaging until use. Prolonged storage at temperatures below **2 °C** creates significant moisture within the material, posing a risk of contamination. Freezing, even momentarily, destroys the material. Excessive heating should also be avoided. The petri dishes can be used until the expiration date indicated on the label. For transportation, our stability studies have shown that the petri dishes can be kept at **6–25 °C** for **4 days** or at **25–40 °C** for **48 hours** without affecting the product's performance.

METHOD OF USE

Inoculate petri dishes using the technique for obtaining isolated colonies.

Incubate at $35-37^{\circ}\text{C}$ under microaerophilic conditions with 5–10% CO_2 and examine the cultures after overnight incubation in the initial phase, and after approximately 48 hours in the final phase.

INTERPRETATION OF RESULTS

BLOOD AGAR:

Streptococcus pyogenes forms small, grayish-white colonies with β -hemolysis.

Streptococcus pneumoniae forms small, flat colonies with α -hemolysis.

Staphylococcus aureus forms relatively large colonies, 1-3 mm after 24 hours of incubation and 3-8 mm if incubation is extended up to 5 days. The color ranges from cream-yellow to orange, depending on the incubation time. It causes either α or β hemolysis.

CHOCOLATE BACITRACIN:

Haemophilus influenzae forms small, moist colonies with a slight odor.

Streptococcus pneumoniae forms small, flat colonies that appear green.


LIMITATIONS OF THE METHOD

Final identification should be done with biochemical and serological tests (e.g., Latex Agglutination Test) and can be performed directly from the suspected colonies.

QUALITY CONTROL

BLOOD AGAR

Microorganism	ATCC	Growth	Hemolysis
<i>Escherichia coli</i>	25922	Good	-
<i>Pseudomonas aeruginosa</i>	27853	Good	-
<i>Staphylococcus aureus</i>	25923	Good	B
<i>Streptococcus pneumoniae</i>	6303	Good	A
<i>Streptococcus pyogenes</i>	19615	A	B

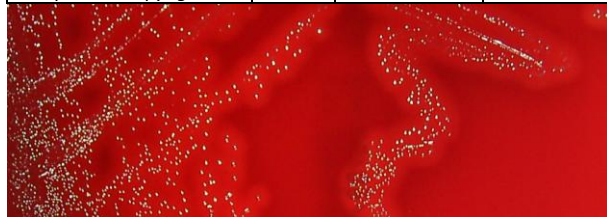Streptococcus pyogenes ATCC 19615

CHOCOLATE BACITRACINE

Microorganism	ATCC	Colonies
<i>Haemophilus influenzae</i>	10211	Small, moist, with a slight odor.
<i>Streptococcus pneumoniae</i>	6305	Small, flat, showing a green color.
<i>Neisseria meningitidis</i>	13090	Partial to complete inhibition.
<i>Neisseria gonorrhoeae</i>	43069	Partial to complete inhibition.

Haemophilus influenzae ATCC 10211

WASTE DISPOSAL OF WASTE

Materials that show no growth can be considered as non-hazardous waste and disposed of accordingly. Materials that show colony growth must be disposed of according to the guidelines for infectious or potentially infectious waste. The laboratory is responsible for the proper management of infectious waste according to its nature and level of risk and must handle and dispose of it (or assign its management and disposal) in compliance with the applicable regulations.

SPECIFICATIONS

BLOOD AGAR – CHOCOLATE BACITRACINE - CE

ITEM	CODE	PACKAGE	STORAGE	SHELF LIFE
Dichotomous petri dish 9cm	020303	10 pieces	2 – 12 °C	2 months

Produced in Greece by the company Bioprep in accordance with the requirements of the European Directive 2017/746.

BASIC UDI-DI: 5212037714010401WF. EDMA (14 01 04 01) non-chromogenic media (Plates).

The Bioprep company has been certified according to the standards: EN ISO 9001:2015 / EAOT EN ISO 13485:2016 DY8d/1348/2004

LITERATURE REFERENCES

BLOOD AGAR

Ellner, P.D., Stoessel, C.J., Drakeford, E. and Vasi, F. (1966). A new culture medium for medical bacteriology. Amer J. Clin Pathol. 45. 502-504.

American Public Health Association (1950). Diagnostic Procedures and Reagents. 3rd edn. A.P.H.A., New York.

CHOCOLATE BACITRACINE

Sng, E.H., Rajan, V.S., and Lim, A.L. (1977) Simplified media for isolating *Neisseria gonorrhoeae* J. Clin. Microbiol., 5 (4), 387.

Sottnek, F. O., Biddle, J. W., Kraus, S. J., Weaver, R. E., And Stewart, J. A. (1980) Isolation and Identification of *Haemophilus ducreyi* in a clinical study. J. Clin. Microbiol., 12 (2), 170.

Lewis, J. S., and Wiesner, P.J. (1980) Gonorrhea: Current laboratory methods. Lab. Management, Sept., p.33.

IN VITRO MANUFACTURER'S DATA

G. PAPANIKOLAOU & CO

PRODUCTION LABORATORIES OF CULTURE MEDIA

Potamou 5, Industrial Area Keratea, Attica

P.O. Box: 4893, Postal Code: 9001 - Tel: +30 2299066113. Fax: +30 2299066112

E-mail: bioprep1@otenet.gr

www.bioprepare.gr