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TECHNICAL DATA SHEET

PRODUCT: **CHROMagar™ Strep A** 
 REFERENCE: **010753 – 050753**

Date 1st Edition:
7th 2009
Date 4th Revision:
6th 2024

DESCRIPTION

CHROMagar™ Strep A is used for the identification of Group A Streptococci (*Streptococcus pyogenes*) in pharyngeal samples.

INTRODUCTION

Infection by *Streptococcus pyogenes* is responsible for nearly all streptococcal cases of bacterial pharyngitis, which makes it the specific target of diagnostic methods for Group A Streptococci. Patients are typically tested for the presence of GAS (Group A Streptococci) using antigen detection tests that provide results within 15–20 minutes, but these have a positive predictive value of ~70–90% and a false negative rate of ~10–20%. However, throat culture is mainly used as a confirmatory test with definitive results that require 24 hours.

PRINCIPLE OF THE METHOD

CHROMagar Strep A is a new chromogenic medium plate that allows for easy reading through the direct color differentiation of GAS colonies (orange to red) among other bacteria from the complex flora of the pharynx (colorless or blue colonies). Meat peptones provide nitrogen, vitamins, minerals, and amino acids essential for growth. Yeast extract is a source of vitamins, particularly from the B group. Sodium chloride supplies essential electrolytes for transport and helps regulate the osmotic balance of the medium. Bacteriological agar is the solidifying agent. Selective agents inhibit the growth of Gram-negative bacteria, fungi, and most Gram-positive cocci.

COMPOSITION	g/litre
Chromogenic mix and selective mix	2,2
Peptones and yeast extract	27,0
Salts	9,0
Growth factors	5,0
Agar	15,0
Selective mix	0,1

Appearance: Clear beige.

Final pH 7.2 ± 0.2 at 25 °C.

PRECAUTIONS

CHROMagar™ Strep A is an in vitro diagnostic laboratory material and must be handled only by qualified laboratory personnel. This product contains peptones and extracts of animal origin. Certificates regarding the origin and health status of the animals do not fully guarantee the absence of transmissible pathogens. Therefore, these materials should be treated as potentially infectious, and standard safety precautions should be followed (they must not be ingested or inhaled). Plates should always be handled with gloves and within a Class II laminar flow hood to avoid contamination, especially from saprophytic fungi. If a plate is cracked or the packaging is damaged, do not use it. Do not use plates that show signs of microbial contamination. The agar layer should be 4–5 mm thick and the material should be free of cracks, dryness, or any other signs of deterioration. After the expiration date, the material is unsuitable for use. In case of contact with skin, wash immediately with plenty of water and soap. Positive samples must be disposed of according to hygiene regulations for the handling of infectious material.

STORAGE AND TRANSPORT CONDITIONS

The plates should be stored at **2–12 °C** in their packaging until use. Prolonged storage below **2 °C** generates excess moisture in the medium, increasing the risk of contamination. Freezing, even momentarily, destroys the medium. Excessive heating should also be avoided. The plates can be used until the expiration date indicated on the label. For transport, our stability studies have shown that the plates can be kept at **14–25 °C** for up to **4 days** or at **27–40 °C** for up to **24 hours** without affecting product performance.

INSTRUCTIONS FOR USE

Inoculate the sample onto one end of the CHROMagar Strep A plate and then perform successive streaks with a loop in parallel lines to isolate individual colonies.

Incubate for 18–24 hours at 35–37 °C aerobically or preferably in 5–10% CO₂.

INTERPRETATION OF RESULTS

Group A Streptococcus (*Streptococcus pyogenes*) forms orange to red colonies.

Enterococcus faecalis form blue colonies.

Other Gram-positive cocci from the pharyngeal flora form colorless or blue colonies.


LIMITATIONS OF THE METHOD

Final identification must be confirmed using biochemical and serological tests (e.g., Latex agglutination test), which can be performed directly from the suspect colonies.

GENERAL CHARACTERISTICS OF QUALITY CONTROL

Microorganism	Growth / Colony Color
<i>Streptococcus pyogenes</i> ATCC® 19615	Orange to red
<i>Enterococcus faecalis</i> ATCC® 29212	Blue

<i>Staphylococcus aureus</i> ATCC® 25923	Inhibited
<i>Escherichia coli</i> ATCC® 25922 (WDCM 00013)	Inhibited
<i>Candida albicans</i> ATCC® 10231	Inhibited

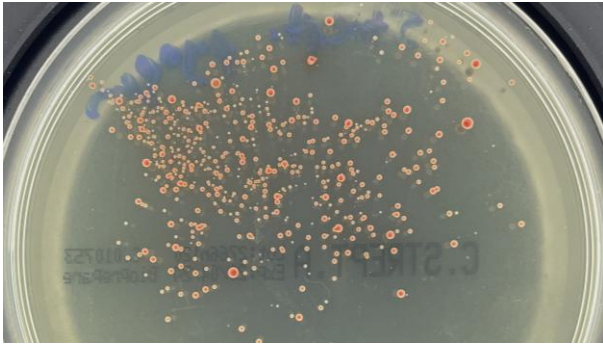Streptococcus pyogenes

WASTE DISPOSAL OF WASTE

Materials that show no growth can be considered as non-hazardous waste and disposed of accordingly. Materials that show colony growth must be disposed of according to the guidelines for infectious or potentially infectious waste. The laboratory is responsible for the proper management of infectious waste according to its nature and level of risk and must handle and dispose of it (or assign its management and disposal) in compliance with the applicable regulations.

SPECIFICATIONS

CHROMagar™ Strep A - CE

ITEM	CODE	PACKAGE	STORAGE	SHELF LIFE
9 cm plate	010753	10 pieces	2 – 12 °C	2 months
6 cm plate	050753	10 pieces	2 – 12 °C	2 months

Produced in Greece by the company Bioprepare in accordance with Regulation (EU) 2017/746. Basic UDI-DI: 5212037714010402WH EDMA: (14 01 04 02) Chromogenic ID Media (Plates) Bioprepare is certified according to the standards: EN ISO 9001:2015 / ELOT EN ISO 13485:2016 / DY8d/1348/2004

REFERENCES

Mark Gaskin et Al. ASM 2019: Validation and implementation of Colorex™ CHROMagar™ Strep A agar on WASP™/WASPLab™ for screening for *Streptococcus pyogenes* using the ESwab™.

Cynthia C. Kayak Alverno Laboratories, Hammond, IN ASM San Francisco 2019:

Digital detection and the use of Artificial Intelligence to detect Group A *Streptococcus* using a chromogenic agar.

Tam T. Van, Kenneth Mata, Jennifer Dien Bard Journal of Clinical Microbiology October 2019: Automated Detection of *Streptococcus pyogenes* Pharyngitis by use of Colorex Strep A CHROMagar and WASPLab Artificial Intelligence Chromogenic Detection Module Software.

IN VITRO MANUFACTURER'S DATA

Bioprepare
microbiology

G. PAPANIKOLAOU & CO

PRODUCTION LABORATORIES OF CULTURE MEDIA

Potamou 5, Industrial Area Keratea, Attica

P.O. Box: 4893, Postal Code: 9001 - Tel: +30 2299066113. Fax: +30 2299066112

E-mail: bioprep1@otenet.gr

www.bioprepare.gr