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American Public Health Association (1950). Diagnostic Procedures and Reagents. 3rd edn. A.P.H.A., New York.

American Public Health Association (1946). Standard Methods for the examination of Water and Sewage. 9th edn. A.P.H.A., New York.

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www.bioprep1.gr

TECHNICAL DATA SHEET

PRODUCT: **COLUMBIA W/ 5% HORSE BLOOD –
MAC CONKEY AGAR No 2**

REFERENCE: **020527**

Date 1st Edition:
7th 2009
Date 4rth Revision:
6th 2024

DESCRIPTION

The divided plate COLUMBIA W/ 5% HORSE BLOOD – MACCONKEY AGAR No. 2 is used for the cultivation of all aerobic bacteria and the identification of hemolytic Gram-positive cocci (COLUMBIA W/ 5% HORSE BLOOD). It is also used for the isolation and identification of Enterobacteriaceae (MACCONKEY AGAR).

PRINCIPLE OF THE METHOD

COLUMBIA HORSE BLOOD AGAR contains highly nutritious ingredients that provide vitamins, carbohydrates, and other organic compounds. Sodium chloride supplies essential minerals and helps maintain osmotic and electrolyte balance. The addition of 5% horse blood enhances the nutrient content. The hemolysis and the type of hemolysis of horse red blood cells assist in identifying certain Gram-positive cocci.

COMPOSITION	g/litre
COLUMBIA W/ 5% HORSE BLOOD	
Columbia peptone mixture	25.1
Soluble starch	1.0
Sodium chloride	5.0
Agar	12.0
Horse Blood Defibrinated	50ml

Appearance: Red to dark red, non-transparent due to the addition of blood.

Final pH 7.3 ± 0.2 at 25 °C.

MACCONKEY AGAR No. 2 is a nutrient medium in which the addition of Bile Salts No. 2 supports the growth not only of Enterobacteria but also of *Enterococci*. Gram-negative enterobacteria that ferment lactose produces red or pink colonies. Bile salts No. 2 and crystal violet inhibit the growth of Gram-positive cocci, except for *Enterococcus*. The pH indicator neutral red changes color with lactose fermentation.

COMPOSITION	g/litre
MAC CONKEY AGAR No. 2	
Peptone	20.0
Lactose	10.0
Bile Salts No. 2	1.5
Sodium chloride	5.0
Neutral red	0.05
Crystal violet	0.001
Agar No. 2	15.0

Appearance: Clear pink to red.

Final pH 7.0 ± 0.2 °C at 25 °C.

PRECAUTIONS

COLUMBIA W/ 5% HORSE BLOOD – MAC CONKEY AGAR No 2 is an in vitro diagnostic laboratory material and should only be handled by trained laboratory personnel. This material contains peptones and extracts of animal origin. Certificates regarding the source and health status of the animals do not fully guarantee the absence of transmissible pathogens. Therefore, these materials should be treated as potentially infectious and handled using standard safety precautions (avoid ingestion and inhalation). Always handle plates wearing gloves and within a Class II laminar flow cabinet to prevent contamination, especially by saprophytic fungi. Do not use plates that show signs of microbial contamination. The material should not be used after the expiration date. In case of skin contact, wash immediately with plenty of water and soap. Positive samples must be disposed of according to the appropriate hygiene regulations for infectious materials.

STORAGE CONDITIONS

The plates should be stored at **2–12 °C** in their original packaging until the time of use. Prolonged storage below **2 °C** creates excess moisture in the medium, increasing the risk of contamination. Even brief freezing destroys the medium. Excessive heating should also be avoided. The plates may be used until the expiration date indicated on the label. For transport, our stability studies have shown that the plates can remain at **6–25 °C** for **4 days** or at **25–40 °C** for **48 hours** without compromising product performance.

INSTRUCTIONS FOR USE

Place the plates in an incubator (35–37 °C) for 30–45 minutes.

Inoculate the sample as soon as possible after collection and streak using serial dilution for isolated colonies.

Incubate under aerobic conditions at 35–37 °C for 24 hours. Refer to the table for colony color and characteristics.

GENERAL QUALITY CONTROL CHARACTERISTICS

COLUMBIA W/ 5% HORSE BLOOD

Microorganism	ATCC	Growth	Hemolysis
<i>Escherichia coli</i>	25922	Good	-
<i>Pseudomonas aeruginosa</i>	27853	Good	-

<i>Staphylococcus aureus</i>	25923	Good	B
<i>Streptococcus pneumoniae</i>	6303	Good	A
<i>Streptococcus pyogenes</i>	19615	Excellent	B

Streptococcus pyogenes ATCC 19615

MAC CONKEY AGAR No 2

Microorganism	ATCC	Growth	Hemolysis
<i>Escherichia coli</i>	25922	Good	Pink
<i>Enterococcus faecalis</i>	29212	Good	Red
<i>Salmonella typhimurium</i>	14028	Good	Colorless
<i>Proteus mirabilis</i>	7002	Good	Colorless
<i>Staphylococci</i>	25923	Excellent	

Escherichia coli

Enterococcus faecalis

WASTE DISPOSAL OF WASTE

Materials that show no growth can be considered as non-hazardous waste and disposed of accordingly. Materials that show colony growth must be disposed of according to the guidelines for infectious or potentially infectious waste. The laboratory is responsible for the proper management of infectious waste according to its nature and level of risk and must handle and dispose of it (or assign its management and disposal) in compliance with the applicable regulations.

SPECIFICATIONS

COLUMBIA W/ 5% HORSE BLOOD – MAC CONKEY AGAR No 2 - 

ITEM	CODE	PACKAGE	STORAGE	SHELF LIFE
Divided petri dish 9cm	020527	10 pieces	2 – 12 °C	2 months

Manufactured in Greece by Bioprep, in compliance with Regulation (EU) 2017/746. Basic UDI-DI: 5212037714010490X8 EDMA Code: (14 01 04 90) Other Prepared Media in Plates Bioprep is certified according to the standards: EN ISO 9001:2015, ELOT EN ISO 13485:2016, and DY86/1348/2004

REFERENCES

Ellener, P.C., C.J. Stoessel, E. Drakeford, and F. Vassi 1966. A new culture medium for medical bacteriology. Am J. Clin Pathol. 45:502-504. Ruoff, k., I. 1995 Streptococcus.
American Public Health Association (1950). Diagnostic Procedures and Reagents. 3rd edn. A.P.H.A., New York.
American Public Health Association (1946). Standard Methods for the examination of Water and Sewage. 9th edn. A.P.H.A., New York.

IN VITRO MANUFACTURER'S DATA

G. PAPANIKOLAOU & CO

PRODUCTION LABORATORIES OF CULTURE MEDIA

Potamou 5, Industrial Area Keratea, Attica

P.O. Box: 4893, Postal Code: 9001 - Tel: +30 2299066113. Fax: +30 2299066112

E-mail: bioprep1@otenet.gr

www.bioprep.gr