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Todd and Hewitt J. Path I Bact. 35:973. 1932 Updyke and Nickle. Applied. Microbiol 2: 117. 1954

Diagnostic Procedures and Reagents. 4th Ed. APHA Inc. New York 1963.

Isenberg H.D. (ed) 1992. Clinical Microbiology procedures handbook, American Society for Microbiology, Washington, D.C. Murray, P.R.,

E. J. Baron, M.A. Pfaller, F.C. Tencver and R.H. Tenckhoff (ed) 1995 Manual of clinical Microbiology, 6th ed. American Society for

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TECHNICAL DATA SHEET

PRODUCT: **TODD – HEWITT BROTH**
 REFERENCE: **080115**

Date 1st Edition:
7th 2009
 Date 4th Revision:
6th 2024

DESCRIPTION

TODD – HEWITT BROTH is used for the cultivation of β -hemolytic Streptococci and their serological identification from clinical samples.

PRINCIPLE OF THE METHOD

The medium is made from a Todd Hewitt base consisting of beef heart, peptone and yeast extract. The basic medium is formulated to provide the necessary nutrients for the development of streptococci. Carbon and energy are provided by glucose. Disodium phosphate and sodium carbonate serve as buffers that ultimately protect haemolysin from acids produced during fermentation of the carbohydrate.

FORMULA	g/litre
Heart Infusion	3.10
Bacteriological Peptone	20.0
Dextrose	2.0
Sodium carbonate	2.5
Sodium chloride	2.0
Disodium phosphate	0.4
Yeast Extract	10.0

Appearance: Beige clear.

Final pH 7.6 ± 0.2 at 25 °C.

PRECAUTIONS

TODD – HEWITT BROTH is an in-vitro laboratory diagnostic material and should only be handled by qualified people in the laboratory. This material contains peptones and extracts of animal origin. The certificates regarding the origin and health status of the animals do not fully guarantee the absence of transmissible pathogens. For this reason, it is recommended that these materials be treated as potentially infectious, with the usual safety precautions (avoiding ingestion or inhalation). Plates should always be handled with gloves and in Laminar flow Class II, to avoid contamination mainly by saprophytic fungi. If the plate is cracked or the bag has a hole, do not use it. Do not use petri dishes if there are signs of microbial contamination. The thickness of the agar must be 4 - 5 mm and the material without cracks, dryness or other signs of deterioration. After the expiry date the material is unfit for use. In case of contact with the skin, wash immediately with plenty of water and soap. Positive samples must be destroyed according to the hygienic rules prescribed for the management of contaminated samples.

STORAGE CONDITIONS

The tubes should be stored at **2 – 12°C** in their original packaging until use. Even momentary freezing can destroy the medium. Excessive heating should also be avoided. The tubes can be used until the expiration date indicated on the label. Stability studies have shown that during transport, the tubes can remain at **6 - 25°C** for up to **5 days** or at **25 - 40°C** for **48 hours** without affecting product performance.

INSTRUCTIONS FOR USE

Inoculate the swab directly into TODD – HEWITT BROTH. Incubate aerobically for 18-24 hours at 35-37°C. After incubation, subculture on COLUMBIA C.N.A. (Sheep Blood). Incubate the plates aerobically for 18-24 hours at 35-37°C.

READING AND INTERPRETATION

Observe the development of β -haemolytic or non-haemolytic, Gram positive colonies in COLUMBIA C.N.A. (Sheep Blood)

Using individual colonies, perform serological tests to detect Streptococcus antigen.

LIMITATIONS OF THE METHOD

TODD – HEWITT BROTH is an enrichment broth. Additional tests, including subculturing on appropriate agar, as well as biochemical or serological tests using pure cultures, must be conducted for full identification. For further information, refer to relevant references.

GENERAL CHARACTERISTICS OF QUALITY CONTROL

Microorganism	Growth
<i>Neisseria meningitidis</i> ATCC 13090	Good
<i>Streptococcus pneumoniae</i> ATCC 6303	Good
<i>Streptococcus pyogenes</i> ATCC 19615	Good

WASTE DISPOSAL OF WASTE

Materials that show no growth can be considered as non-hazardous waste and disposed of accordingly. Materials that show colony growth must be disposed of according to the guidelines for infectious or potentially infectious waste. The laboratory is responsible for the proper management of infectious waste according to its nature and level of risk and must handle and dispose of it (or assign its management and disposal) in compliance with the applicable regulations.

SPECIFICATIONS

TODD – HEWITT BROTH - 

PRODUCT	CODE	PACKING	STORE	SELF LIFE
Tube 2ml	080115	20 pieces	2 – 12 °C	12 months

Produced in Greece by Bioprepate in accordance with the requirements of the European Directive 98/79/EK. FEK B2198/2-10-2009. CODE EDMA 14 01 02 01. Bioprepate company has been certified according to standards EN ISO 9001:2015 / EAOT EN ISO 13485:2016 DY8d/1348/2004

BIBLIOGRAPHY

Todd and Hewitt J. Path I Bact. 35:973. 1932 Updyke and Nickle. Applied. Microbiol 2: 117. 1954
Diagnostic Procedures and Reagents. 4th Ed. APHA Inc. New York 1963.

Isenberg H.D. (ed) 1992. Clinical Microbiology procedures handbook, American Society for Microbiology, Washington, D.C. Murray, P.R.,
E. J. Baron, M.A. Pfaller, F.C. Tencver and R.H. Tenk (ed) 1995 Manual of clinical Microbiology, 6th ed. American Society for
Microbiology, Washington, D.C. MacFaddin, J.F. (1983). Biochemical Tests for Identification of Medical Bacteria. Williams and Wilkins.

IN VITRO MANUFACTURER'S DATA

G. PAPANIKOLAOU & CO

PRODUCTION LABORATORIES OF CULTURE MEDIA

Potamou 5, Industrial Area Keratea, Attica

P.O. Box: 4893, Postal Code: 9001 - Tel: +30 2299066113. Fax: +30 2299066112

E-mail: bioprep1@otenet.gr www.bioprepate.gr