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American Public Health Association (1950). Diagnostic Procedures and Reagents. 3rd edn. A.P.H.A., New York.

MAC CONKEY AGAR

American Public Health Association (1950). Diagnostic Procedures and Reagents. 3rd edn. A.P.H.A., New York.

American Public Health Association (1946). Standard Methods for the examination of Water and Sewage. 9th edn. A.P.H.A., New York.

CHROMagar™ CANDIDA

CHROMagar Candida, a new differential isolation medium for the presumptive identification of clinically important Candida species.

Odds F. et al.

1994, Journal of Clinical Microbiology, 32 : 1923-1929. Use of CHROMagar Candida medium for isolation of yeasts from dental samples.

Beighton D. et al.

1995. Journal of Clinical Microbiology, 33 : .3025-3027.

Application of CHROMagar Candida for rapid screening of clinical specimens for *Candida albicans*, *Candida tropicalis*, *Candida krusei*, and *Candida (Torulopsis) glabrata*.

Pfaller M.A. et al.

1996. Journal of Clinical Microbiology, 34 : 58-61. Routine use of CHROMagar Candida medium for presumptive identification of *Candida* yeast species and detection of mixed fungal populations.

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6th 2024

DESCRIPTION

Tripartite petri dish BLOOD AGAR – MAC CONKEY AGAR – CHROMagar™ CANDIDA for the cultivation of all aerobic bacteria and identification of hemolytic gram (+) cocci (BLOOD AGAR) – For the cultivation, isolation, and identification of enterobacteria (Enterococcus does not grow) (MAC CONKEY AGAR) – For selective isolation and chromatic differentiation of Candida fungi (CHROMagar™ CANDIDA).

METHOD PRINCIPLE

The Blood Agar Base contains high-nutrient components that provide vitamins, carbohydrates, and other organic elements. Sodium chloride supplies essential minerals and maintains osmotic balance and electrolyte equilibrium. The addition of 6% horse blood enhances the nutritional value of the medium. Furthermore, the hemolysis and type of hemolysis of horse red blood cells assist in the identification of certain gram (+) cocci.

FORMULA BLOOD AGAR	g/litre
Columbia Peptone Mixture	25.1
Corn Starch	1.0
Sodium chloride	5.0
Agar No. 2	12.0
Horse Blood	60ml

Appearance: Red – burgundy, non-transparent, due to the addition of blood.

Final pH 7.3 ± 0.2 in 25 °C.

MAC CONKEY AGAR is a nutrient medium used for the isolation of enterobacteria. Gram-negative enterobacteria that ferment lactose produces red or pink colonies. Bile salts No. 3 and crystal violet inhibit the growth of gram-positive cocci. The pH indicator, neutral red, changes color with the breakdown of lactose.

FORMULA MAC CONKEY AGAR	g/litre
Peptone	20.0
Lactose	10.0
Bile Salts No. 3	1.5
Sodium chloride	5.0
Neutral red	0.03
Crystal violet	0.001
Agar No. 2	15.0

Appearance: Clear pink-purple.

Final pH 7.1 ± 0.2 in 25°C.

In CHROMagar™ CANDIDA, meat peptones provide nitrogen, vitamins, minerals, and amino acids essential for growth. The yeast extract is the source of vitamins, particularly the B group. Sodium chloride provides essential electrolytes for transport and regulates the osmotic balance of the material. Chloramphenicol is a broad-spectrum antibiotic that inhibits a large group of gram-positive and gram-negative bacteria. The bacteriological agar is the solidifying agent.

FORMULA CHROMagar™ CANDIDA	g/litre
Peptone	10.2
Chromogenic mix	22.0
Chloramphenicol	0.5
Agar	15.0

Appearance: Light beige transparent

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To BLOOD AGAR – MAC CONKEY AGAR – CHROMagar™ CANDIDA is an in-vitro laboratory diagnostic material and should only be handled by qualified people in the laboratory. This material contains peptones and extracts of animal origin. The certificates regarding the origin and health status of the animals do not fully guarantee the absence of transmissible pathogens. For this reason, it is recommended that these materials be treated as potentially infectious, with the usual safety precautions (avoiding ingestion or inhalation). Plates should always be handled with gloves and in Laminar flow Class II, to avoid contamination mainly by saprophytic fungi. If the plate is cracked or the bag has a hole, do not use it. Do not use petri dishes if there are signs of microbial contamination. The thickness of the agar must be 4 - 5 mm and the material without cracks, dryness or other signs of deterioration. After the expiry date the material is unfit for use. In case of contact with the skin, wash immediately with plenty of water and soap. Positive samples must be destroyed according to the hygienic rules prescribed for the management of contaminated samples.

STORAGE AND TRANSPORT CONDITIONS

The petri dishes must be stored at **2 – 12 °C** inside their packaging until they are used. Prolonged storage at temperatures below **2 °C** creates excessive moisture inside the material, which poses a risk of contamination. Freezing, even momentarily, destroys the material. Excessive

heating should also be avoided. The petri dishes can be used until the expiration date indicated on the label. For transport, our stability studies have shown that the petri dishes can remain at **6 - 25 °C for 4 days** or at **25 - 40 °C for 48 hours** without affecting the product's performance.

INSTRUCTIONS FOR USE

Plate the petri dishes using the technique of individual colony picking.
Incubate them at 35 – 37 °C for 24 hours under aerobic conditions.

INTERPRETATION OF RESULTS

BLOOD AGAR:

Streptococcus pyogenes forms small, gray-white colonies with beta-hemolysis.

Streptococcus pneumoniae forms small, flat colonies with alpha-hemolysis.

Staphylococcus aureus forms relatively large colonies, 1-3 mm after 24 hours of incubation and 3-8 mm if the incubation is extended up to 5 days. Their color ranges from cream-yellow to orange, depending on the incubation time. It also causes alpha or beta hemolysis.

MAC CONKEY AGAR:

On MAC CONKEY AGAR, gram (+) cocci do not grow.

Gram (-) bacteria that ferment lactose produce pink to red colonies due to acid production. Additionally, *Escherichia coli* produces pink or red colonies surrounded by a cloudy zone, resulting from the precipitation of bile salts due to the pH drop.

Gram (-) bacteria that do not ferment lactose produce colorless colonies. *Proteus* does not swarm.

CHROMagar™ CANDIDA:

C. Albicans forms colonies with a green color.

C. tropicalis forms colonies with a metallic blue color.

C. krusei forms colonies that are pink and undefined.

C. kefyr & *C. glabrata* form colonies with a purple to brown color.

Other Candida strains form colonies with a white to pink color.


LIMITATIONS OF THE METHOD:

Final identification must be confirmed with biochemical and serological tests (e.g., latex agglutination test), which can be performed directly from the suspected colonies. In CHROMagar™ CANDIDA, weak or normal growth of Gram-positive and Gram-negative bacteria may occur when their count in the inoculum is $\geq 10^6$ CFU/ml.

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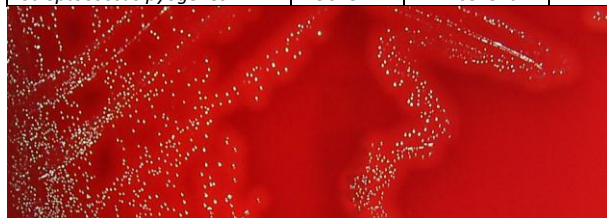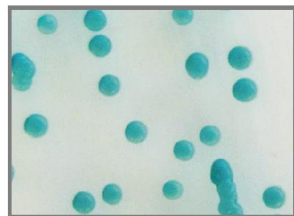
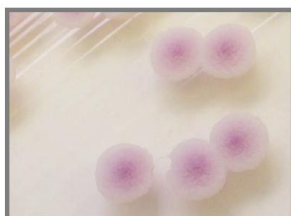
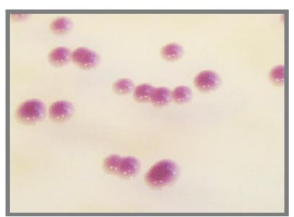
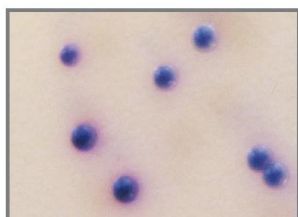
C.tropicalis

WASTE DISPOSAL OF WASTE

Materials that show no growth can be considered as non-hazardous waste and disposed of accordingly. Materials that show colony growth must be disposed of according to the guidelines for infectious or potentially infectious waste. The laboratory is responsible for the proper management of infectious waste according to its nature and level of risk and must handle and dispose of it (or assign its management and disposal) in compliance with the applicable regulations.

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Produced in Greece by the company Bioprep in accordance with the requirements of the European Directive 2017/746.

BASIC UDI-DI: 5212037714010401WF. EDMA (14 01 04 01) non-chromogenic media (Plates).

The Bioprep company has been certified according to the standards: EN ISO 9001:2015 / EAOT EN ISO 13485:2016 DY8d/1348/2004

LITERATURE REFERENCES

BLOOD AGAR

Ellner, P.D., Stoessel, C.J., Drakeford, E. and Vasi, F. (1966). A new culture medium for medical bacteriology. Amer J. Clin Pathol. 45. 502-504.

American Public Health Association (1950). Diagnostic Procedures and Reagents. 3rd edn. A.P.H.A., New York.

MAC CONKEY AGAR

American Public Health Association (1950). Diagnostic Procedures and Reagents. 3rd edn. A.P.H.A., New York.

American Public Health Association (1946). Standard Methods for the examination of Water and Sewage. 9th edn. A.P.H.A., New York.

CHROMagar™ CANDIDA

CHROMagar Candida, a new differential isolation medium for the presumptive identification of clinically important Candida species.

Odds F. et al.

1994, Journal of Clinical Microbiology, 32 : 1923-1929. Use of CHROMagar Candida medium for isolation of yeasts from dental samples.

Beighton D. et al.

1995. Journal of Clinical Microbiology, 33 : 3025-3027.

Application of CHROMagar Candida for rapid screening of clinical specimens for Candida albicans, Candida tropicalis, Candida krusei, and Candida (Torulopsis) glabrata.

Pfaller M.A. et al.

1996. Journal of Clinical Microbiology, 34 : 58-61. Routine use of CHROMagar Candida medium for presumptive identification of Candida yeast species and detection of mixed fungal populations.

Bouchara J.P. et al.

1996. Technical Report, 202-208..

IN VITRO MANUFACTURER'S DATA

G. PAPANIKOLAOU & CO

PRODUCTION LABORATORIES OF CULTURE MEDIA

Potamou 5, Industrial Area Keratea, Attica

P.O. Box: 4893, Postal Code: 9001 - Tel: +30 2299066113. Fax: +30 2299066112

E-mail: bioprep1@otenet.gr

www.bioprep.gr