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6th 2024

DESCRIPTION

For the cultivation, isolation, and identification of Enterobacteria. (Enterococcus does not grow).

PRINCIPLE OF THE METHOD

MAC CONKEY AGAR is a nutrient medium for the isolation of Enterobacteria. Gram-negative enterobacteria that ferment lactose produces red or pink colonies. Bile salts No. 3 and crystal violet inhibit the growth of gram-positive cocci. The neutral red indicator changes colour in response to lactose fermentation.

FORMULA	g/litre
Peptone	20.0
Lactose	10.0
Bile Salts No. 3	1.5
Sodium chloride	5.0
Neutral red	0.03
Crystal violet	0.001
Agar No. 2	15.0

Appearance: Pink - purple clear.

Final pH 7.1 ± 0.2 at 25 °C.

PRECAUTIONS

MAC CONKEY AGAR is an in-vitro laboratory diagnostic material and should only be handled by qualified people in the laboratory. This material contains peptones and extracts of animal origin. The certificates regarding the origin and health status of the animals do not fully guarantee the absence of transmissible pathogens. For this reason, it is recommended that these materials be treated as potentially infectious, with the usual safety precautions (avoiding ingestion or inhalation). Plates should always be handled with gloves and in Laminar flow Class II, to avoid contamination mainly by saprophytic fungi. If the plate is cracked or the bag has a hole, do not use it. Do not use petri dishes if there are signs of microbial contamination. The thickness of the agar must be 4 - 5 mm and the material without cracks, dryness or other signs of deterioration. After the expiry date the material is unfit for use. In case of contact with the skin, wash immediately with plenty of water and soap. Positive samples must be destroyed according to the hygienic rules prescribed for the management of contaminated samples.

STORAGE CONDITIONS

Petri dishes should be stored at **2–12°C** in their packaging until ready for use. Prolonged storage at temperatures below **2°C** can cause moisture to accumulate in the medium, increasing the risk of contamination. Freezing, even momentarily, destroys the medium. Avoid excessive heating as well. Petri dishes can be used until the expiration date indicated on the label. For transport, stability studies have shown that the dishes can remain at **6–25°C** for **10 days** or at **25–40°C** for **72 hours** without affecting product performance.

USAGE

Spread the petri dishes using the streaking technique to isolate single colonies.

Incubate them at 35–37°C for 24 hours under aerobic conditions.

GENERAL CHARACTERISTICS OF QUALITY CONTROL

Microorganism	ATCC	Growth	Colony Color
<i>Salmonella typhimurium</i>	14028	Good	Colorless - translucent
<i>P. mirabilis</i>	12453	Good	Colorless
<i>Escherichia coli</i>	25922	Acceptable	Pink or red surrounded by a blurry zone
<i>Enterococcus faecalis</i>	29212	Inhibited	

Escherichia coli ATCC 25922

P. mirabilis ATCC 12453

WASTE DISPOSAL OF WASTE

Materials that show no growth can be considered as non-hazardous waste and disposed of accordingly. Materials that show colony growth must be disposed of according to the guidelines for infectious or potentially infectious waste. The laboratory is responsible for the proper management of infectious waste according to its nature and level of risk and must handle and dispose of it (or assign its management and disposal) in compliance with the applicable regulations.

SPECIFICATIONS

MAC CONKEY AGAR - GR/CA01/GRM5/O/5 - 

PRODUCT	CODE	PACKING	STORAGE	SELF LIFE
Plate 9cm	010065	10 pieces	2 – 12 °C	5 months
Plate 6cm	050065	10 pieces	2 – 12 °C	5 months
Bottle 100ml	060065	50 pieces	2 – 25 °C	12 months
Tube 10ml	070065	40 pieces	2 – 12 °C	12 months

Produced in Greece by Bioprepare in accordance with the requirements of the European Directive 98/79/EK. FEK B2198/2-10-2009. CODE EDMA 14 01 04 01. Bioprepare company has been certified according to standards EN ISO 9001:2015 / EAOT EN ISO 13485:2016 DY8d/1348/2004

BIBLIOGRAPHY

American Public Health Association (1950). Diagnostic Procedures and Reagents. 3rd edn. A.P.H.A., New York.

American Public Health Association (1946). Standard Methods for the examination of Water and Sewage. 9th edn. A.P.H.A., New York.

IN VITRO MANUFACTURER'S DATA

G. PAPANIKOLAOU & CO

PRODUCTION LABORATORIES OF CULTURE MEDIA

Potamou 5, Industrial Area Keratea, Attica

P.O. Box: 4893, Postal Code: 9001 - Tel: +30 2299066113. Fax: +30 2299066112

E-mail: bioprep1@otenet.gr

www.bioprepare.gr