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TECHNICAL DATA SHEET

PRODUCT: **CHROMagar™ ORIENTATION**
 REFERENCE: **010030 - 050030**

Date 1st Edition:
7th 2009
 Date 4th Revision:
6th 2024

DESCRIPTION

CHROMagar™ Orientation is a non-selective chromogenic culture medium intended for use in the qualitative direct detection, differentiation and presumptive identification of uropathogens to aid in the diagnosis of urine tract infections. The test is performed with urine samples. Results can be interpreted after 18-24 h of aerobic incubation at 35-37 °C

PRINCIPLE OF THE METHOD

Meat peptones provide nitrogen, vitamins, minerals, and amino acids essential for growth. Yeast extract serves as a source of vitamins, particularly from the B group. Chromogenic substrates are broken down by specific microbial enzymes, releasing insoluble colored compounds that attach to the bacterial cell membrane. Sodium chloride provides essential electrolytes for transport and regulates the osmotic balance of the medium. Bacteriological agar acts as the solidifying agent.

FORMULA	g/litre
Chromogenic mix	1,0
Peptone, yeast and extract	17,0
Agar	15,0

Appearance: Light beige, transparent

Final pH 7.0 ± 0.2 at 25 °C.

PRECAUTIONS

CHROMagar™ ORIENTATION is an in-vitro laboratory diagnostic material and should only be handled by qualified people in the laboratory. This material contains peptones and extracts of animal origin. The certificates regarding the origin and health status of the animals do not fully guarantee the absence of transmissible pathogens. For this reason, it is recommended that these materials be treated as potentially infectious, with the usual safety precautions (avoiding ingestion or inhalation). Plates should always be handled with gloves and in Laminar flow Class II, to avoid contamination mainly by saprophytic fungi. If the plate is cracked or the bag has a hole, do not use it. Do not use petri dishes if there are signs of microbial contamination. The thickness of the agar must be 4 - 5 mm and the material without cracks, dryness or other signs of deterioration. After the expiry date the material is unfit for use. In case of contact with the skin, wash immediately with plenty of water and soap. Positive samples must be destroyed according to the hygienic rules prescribed for the management of contaminated samples.

STORAGE CONDITIONS

Petri dishes should be stored at **2–12 °C** in their packaging until use. Prolonged storage below **2 °C** creates excess moisture within the medium, increasing the risk of contamination. Even brief freezing destroys the medium. Excessive heating should also be avoided. Petri dishes can be used until the expiration date printed on the label. If you open the airtight packaging and have leftover plates, store them in the bag until the expiration date. Stability studies have shown that for transport, plates can remain at **6–25 °C** for **5 days** or at **25–40 °C** for **48 hours** without affecting product performance.

USAGE

The infectious material must reach the laboratory quickly without delay and be protected from excessive heat and cold. If processing is delayed, the sample should be inoculated into an appropriate transport medium and stored at 2–12 °C until inoculation. Allow the petri dishes to warm to room temperature. The agar surface must be dry before inoculation. Inoculate the sample at one edge of the plate and then streak it in parallel lines using a loop to obtain isolated colonies. Incubate the plates aerobically at 35–37 °C for 18 to 24 hours.

INTERPRETATION OF RESULTS

Microorganism	Typical colony appearance
Gram (-)	
<i>Escherichia coli</i>	dark pink to reddish
<i>Klebsiella, Enterobacter, Citrobacter, Serratia</i>	metallic blue (+/- reddish halo)
<i>Proteus, Morganella, Providencia</i>	brown halo
<i>Proteus vulgaris</i>	blue with brown halo
<i>Pseudomonas</i>	translucent (+/- natural pigmentation cream to green)
<i>Acinetobacter</i>	cream
<i>Stenotrophomonas</i>	colourless
Gram(+)	
<i>S. agalactiae</i>	light blue
<i>Enterococcus</i>	turquoise blue
<i>S. aureus</i>	golden, opaque, small
<i>S. epidermidis</i>	cream, pinpoint colonies
<i>S. saprophyticus</i>	pink, opaque, small
Yeasts	
<i>Candida albicans</i>	cream, pinpoint colonies

LIMITATIONS

Certain strains of *E. coli* may exhibit pale pink to colourless colonies. Final identification may require additional tests, such as biochemical or immunological assays (latex agglutination).

GENERAL CHARACTERISTICS OF QUALITY CONTROL

Microorganism	Typical colony appearance
<i>Escherichia coli</i> ATCC® 25922	reddish
<i>Enterococcus faecalis</i> ATCC® 29212	turquoise blue
<i>Staphylococcus aureus</i> ATCC® 25923	golden yellow
<i>Staphylococcus saprophyticus</i> ATCC® 15305	pink
<i>S. epidermidis</i> ATCC® 12228	colourless
<i>K. pneumoniae</i> ATCC® 13883	metallic blue


Escherichia coli


Enterococcus faecalis

Proteus mirabilis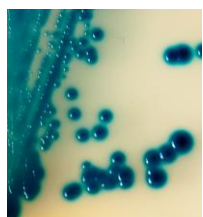


Klebsiella pneumoniae

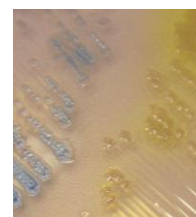
Pseudom. Aeruginosa

S. aureus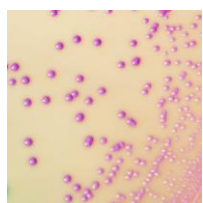

S. Saprophyticus

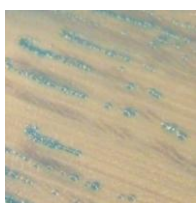S. agalactiae

Candida albicans

WASTE DISPOSAL OF WASTE

Materials that show no growth can be considered as non-hazardous waste and disposed of accordingly. Materials that show colony growth must be disposed of according to the guidelines for infectious or potentially infectious waste. The laboratory is responsible for the proper management of infectious waste according to its nature and level of risk and must handle and dispose of it (or assign its management and disposal) in compliance with the applicable regulations.

SPECIFICATIONS

CHROMagar ORIENTATION - GR/CA01/GRM5/O/3 - CE

PRODUCT	CODE	PACKING	STORE	SELF LIFE
Plate 9cm	010030	10 pieces	2 – 12 °C	5 months
Plate 6cm	050030	10 pieces	2 – 12 °C	5 months

Produced in Greece by Bioprepure in accordance with the requirements of the European Directive 98/79/EK. FEK B2198/2-10-2009. CODE EDMA 14 01 04 02. Bioprepure company has been certified according to standards EN ISO 9001:2015 / EAOT EN ISO 13485:2016 DY8d/1348/2004.

BIBLIOGRAPHY

(1) Merlino, J. et al. 1996. Evaluation of CHROMagar Orientation for Differentiation and Presumptive Identification of Gram-Negative Bacilli and Enterococcus Species, J.C.M. 34: 1788-1793. (2) Samra, Z. et al. 1998. Evaluation of Use of a New Chromogenic Agar in Detection of Urinary Tract Pathogens. J.C.M. 36: 990-994.

IN VITRO MANUFACTURER'S DATA

Bioprepare
microbiology

G. PAPANIKOLAOU & CO

PRODUCTION LABORATORIES OF CULTURE MEDIA

Potamou 5, Industrial Area Keratea, Attica

P.O. Box: 4893, Postal Code: 9001 - Tel: +30 2299066113. Fax: +30 2299066112

E-mail: bioprep1@otenet.gr www.bioprepare.gr